


# FREEDOM60<sup>®</sup>

Syringe Infusion System

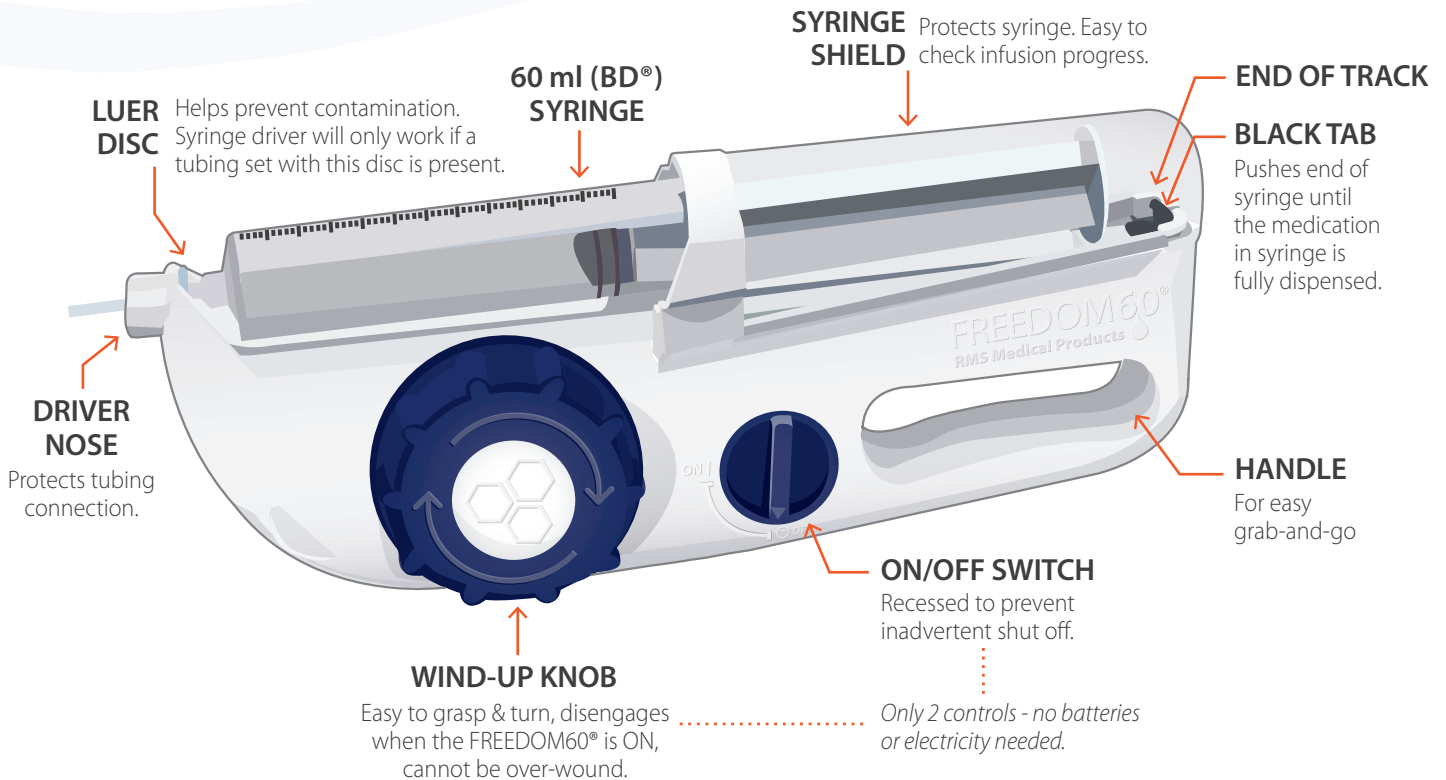


## Indications For Use:

The FREEDOM60<sup>®</sup> Syringe Infusion System is indicated for the infusion of prescribed liquid medicines.

## Infusions made easy as 1, 2, 3...

### 1 Wind

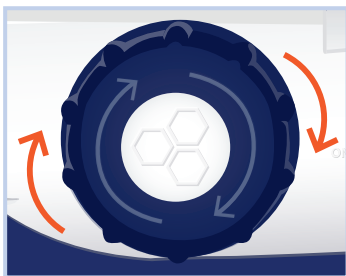


Wind knob until black tab is at end of its track.

### 2 Load



Load prepared syringe.

### 3 Go

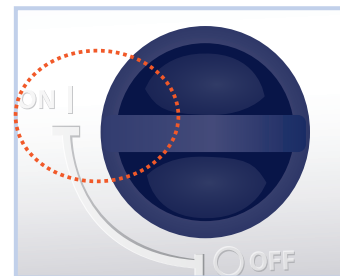

Switch ON and go!



[rmsmedicalproducts.com](http://rmsmedicalproducts.com) 845-469-2042 800-624-9600

The FREEDOM60<sup>®</sup> and FreedomEdge<sup>®</sup> Syringe Infusion Systems, RMS Precision Flow Rate Tubing<sup>™</sup>, and HigH-Flo Subcutaneous Safety Needle Sets<sup>™</sup> are registered trademarks of RMS Medical Products and are compliant with Medical Device Directive 93/42/EEC. RMS Medical Products is ISO 13485 certified. © 2017 RMS Medical Products; All Rights Reserved. F60\_FL\_TechSheet\_03-1126vB

# FREEDOM60<sup>®</sup>

## Syringe Infusion System

### Easy to Use:

- Only two operating knobs:
  - *Large, ergonomic wind knob is easy to grasp and turn*
  - *Recessed ON/OFF switch*
- Just 3 steps – wind, load and go
- Completely portable:
  - *No batteries or electric needed*
  - *Travel pouch included for easy transport*
- No alarms or product maintenance

### Integrated Safety:

- Maintains safe pressure of 13.5 psi (0.93 bar)
- Only RMS Freedom systems have DynEQ™ (Dynamic Equilibrium), which:
  - *Automatically responds to increasing pressure (caused by tissue saturation) by slowing down the flow rate during delivery*
- Holds full pressure after infusion is complete to prevent blood return or drug backflow
- Eliminates:
  - *Concerns of a bolus*
  - *Overflow*
  - *Overdose or runaway infusion*

### Quality, Rugged Design:

- Made in the USA with highest level of quality control for defect-free product
- Robust design made with molded ABS plastic housing
- No sensitive electric parts
- Reliable system, consistent infusions

### Versatile:

- Can be used for various administrations
- RMS Precision tubing sets from F0.5-F2400

## Ordering Information

### Product Description:

FREEDOM60<sup>®</sup> Syringe Driver with Travel Pouch

Replacement Travel Pouch - black

Zebra Print Travel Pouch - pattern

### Item #:

F10050

F10090

F10080



## Technical Specs

### Syringe Driver:

Weight: 14 oz (0.4 kg)  
 Length: 12" (304 mm)  
 Width: 4.5" (114 mm)  
 Height: 1.6" (41 mm)

### System:

Accuracy: +/- 8%  
 Residual volume: <0.4 ml  
 Max. operating pressure: 15 psi (1.03 bar)

### Syringe:

Reservoir volume: 60 ml (max)  
 60 ml BD<sup>®</sup> or Monoject<sup>®</sup> syringes

## Other Products:



**FreedomEdge<sup>®</sup> Syringe Infusion System**  
 for smaller infusion doses, 20 ml and 30 ml syringes



**RMS Precision Flow Rate Tubing<sup>™</sup>**  
 for flow rates from F0.5 - F2400



**High-Flo Subcutaneous Safety Needle Sets<sup>™</sup>**  
 available in 24G and 26G, 4 mm - 14 mm lengths and 1-6 needle leg sets



**24" Extension Tubing Set (FEXT)**  
 for those who need a longer length



[rmsmedicalproducts.com](http://rmsmedicalproducts.com)

845-469-2042

800-624-9600

The FREEDOM60<sup>®</sup> and FreedomEdge<sup>®</sup> Syringe Infusion Systems, RMS Precision Flow Rate Tubing<sup>™</sup>, High-Flo Subcutaneous Safety Needle Sets<sup>™</sup>, and DynEQ<sup>™</sup> are registered trademarks of RMS Medical Products and are compliant with Medical Device Directive 93/42/EEC. RMS Medical Products is ISO 13485 certified. © 2017 RMS Medical Products; All Rights Reserved. F60\_FL\_TechSheet\_03-1126vB